# Lok-Bar™

REFERENCE GUIDE

---

**CIVCO**  
Radiotherapy™

CE

<p style="text-align: center;">Prodigy™ Lok-Bar™</p>	<p style="text-align: center;">Interloc® Lok-Bar™</p>
<p style="text-align: center;">Exact® Lok-Bar™</p>	<p style="text-align: center;">Lok-Bar™</p>
<p style="text-align: center;">Slide Guide Lok-Bar™</p>	<p style="text-align: center;">Rail Mount Lok-Bar™</p>
<p style="text-align: center;">PinnPoint™ Lok-Bar™</p>	<p style="text-align: center;">T-pin Lok-Bar™</p>

<p>I. Overlay          II. Lok-Bar™          III. Lok-Bar™ Pins</p>	<p>IV. Prodigy™ Buttons          V. Interloc® notches          VI. Camlok</p>	<p>VII. Exact® Notches          VIII. T-Pin          IX. Body Pro-Lok™ Platform</p>
---	---	---

**INTENDED USE**

The Lok-Bar™ is a component of an indexing system on the table top which allows for positioning of accessories.

**CAUTION**

Federal (United States) law restricts this device to sale by or on the order of a physician.

**⚠ WARNING**

- Do not use if device appears damaged.
- Exact® Lok-Bar™: Regularly inspect o-rings to ensure device fits properly.
- Do not use to secure positioning devices.
- Do not use device with unapproved accessories or devices.
- Verify all angles of treatment, attenuation characteristics and WET values prior to treating patients. Refer to Technical Data Sheet at [WWW.CIVCORT.COM](http://WWW.CIVCORT.COM) for WET values.

NOTE: If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within the European Union, also report to the competent authority of the Member State in which you are established.

**USING Lok-Bar™**

1. Snap Lok-Bar™ at appropriate longitudinal position.


**⚠ WARNING**

- Ensure Lok-Bar™ is secure.

2. Place positioning accessory onto Lok-Bar™ pins.

**USING COUCHTOP FIXATION**

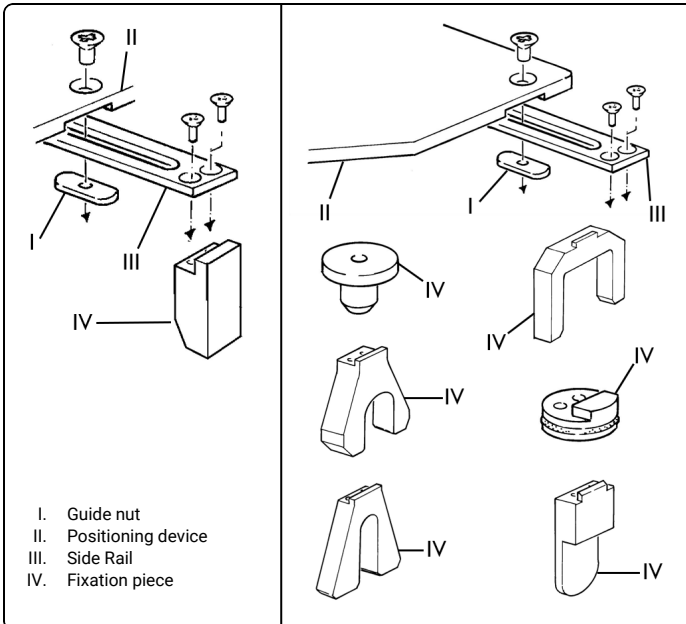1. Attach side rail to positioning device.
2. Attach desired fixation piece to side rail.

## REPROCESSING

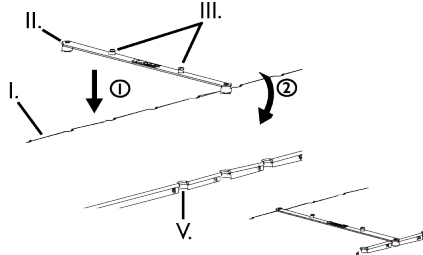
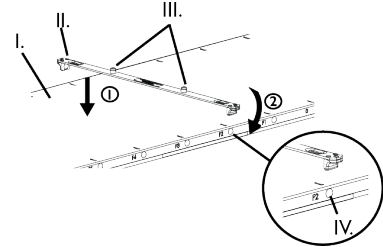
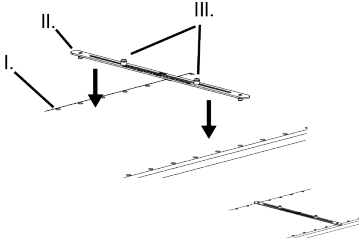
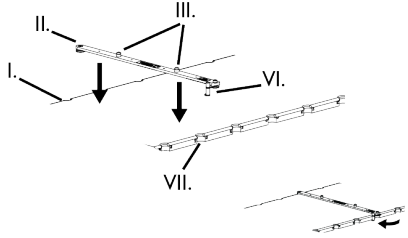
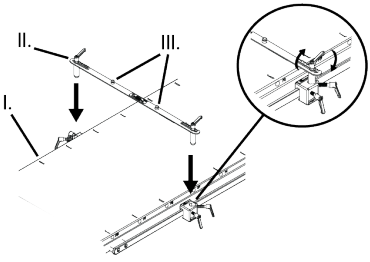
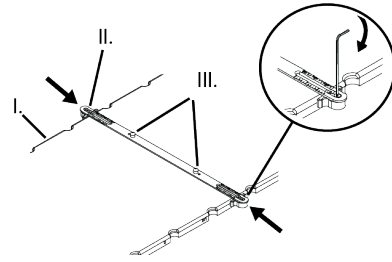
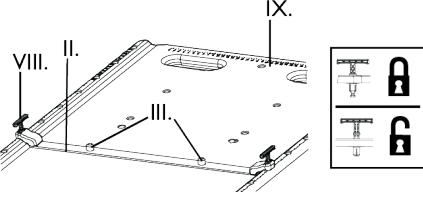
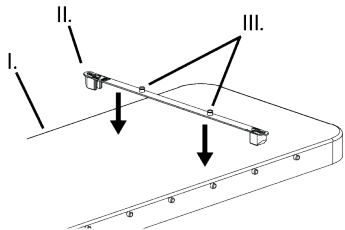
### **WARNING**

- *Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.*

1. Disinfect with a mild, non-abrasive cleanser.
2. Wipe dry with lint-free cloth or allow to air dry prior to reassembly and use.

## MAINTENANCE

NOTE: Inspect device prior to use for signs of damage and general wear.

<p>™Interloc® Lok-Bar</p> 	<p>™Prodigy™ Lok-Bar</p> 
<p>™Lok-Bar</p> 	<p>™Exact® Lok-Bar</p> 
<p>وحدة تثبيت قضيب Lok-Bar™</p>	<p>موجه انزلاق Lok-Bar™</p>
	
<p>مسمار بشكل حرف T Lok-Bar™</p>	<p>™PinnPoint™ Lok-Bar</p>
	

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Radiotherapy™

 CIVCO RADIOTHERAPY

1401 8th Street SE  
Orange City, IA 51041  
United States

800.842.8688, +1 712.737.8688  
[info@civcort.com](mailto:info@civcort.com)

---

COPYRIGHT © 2023 ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. CIVCO RADIOTHERAPY AND CIVCO RT ARE TRADEMARKS OF MEDTEC LLC. INTERLOC AND EXACT ARE REGISTERED TRADEMARKS OF MEDTEC LLC. BODY PRO-LOK, LOK-BAR, PINNPOINT AND PRODIGY ARE TRADEMARKS OF MEDTEC LLC. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA.

[WWW.CIVCORT.COM](http://WWW.CIVCORT.COM)